

DST8000 Maintenance

A yearly inspection is recommended. Note - configuration and colors may differ, depending on your model and accessories.

1. Visual inspection

1. Make sure that the ceiling height is at least 2.6 meter.
2. Make sure that no objects or other equipment is touching the DST or too close to it.
3. Both sides of the DST must be spacious enough to allow physiotherapists to move freely around the device to adjust the handrails. A clearance of 50 cm is recommended at both sides of the DST.
4. Make sure that there are no windows near the DST in a way that an opened window will interfere with the up movement of the DST
5. To avoid water splash, make sure that there is no source of water, like a sink, near the DST.
6. Make sure that no objects, like physiotherapy resistance band (Therabands) are tied to the DST.
7. If after the above visual inspection, relocation of the DST is needed, please refer to the attached **DST User Guide**.

2. Raise the DST to the highest position

1. The DST should go up smoothly and quietly.
2. Pull out the electric plug out of the wall outlet.
3. Inspect the integrity of the main electric cord of the unit and of the cord of the hand control.

3. Physical inspection

1. Fig #1 - make sure these bolts are tightened. Tighten if needed. Use 4 mm Allen key and 10 mm Wrench key (supplied with the DST).
 2. Fig #2 - make sure that the safety pins are in place. If not, find replacements and inset them. Do not use the DST without these safety pins.
 3. Fig #3 – make sure these bolts and nuts are tightened. Tighten if needed. Use 5 mm Allen key and 13 mm Wrench key (supplied with the DST).
 4. Fig #4 – make sure these bolts are tightened. Tighten if needed. Use 4 mm Allen key (supplied with the DST).
 5. Fig #5 – make sure that the leveling feet touch the floor.
 6. Fig #6 - make sure these bolts and nuts are tightened. Tighten if needed. Use 5 mm Allen key and 13 mm Wrench key (supplied with the DST).
 7. Fig #7 - make sure these bolts and nuts are tightened. Tighten if needed. Use 4 mm Allen key and 10 mm Wrench key (supplied with the DST).
 8. Adjust the height of the handrails. See instructions at the attached **DST User Guide** page #5 (adjusting the handrail height). Make sure that the handrails are stable at all heights.
 9. Note: you may have to remove decorative covers from nuts before tightening.
4. Document your inspection and send a copy to info@dpemed.com . The report will enable us to advise you regarding the condition of your DST and to give you further instructions if needed.

