



# TR250T WHEEL CHAIR STAIR CLIMBER

Electric Drive/Track Drive/Easy to Operate





# Product Structure

**Emergency Stop Switch**

**Up and Down Button**

**Angle Display**

**Run Button**

**Handler**

Activate the switch to control the operation of the equipment

**Headrest**

Conforms to the contours of the head, dampens significantly and is comfortable to lean on

**Adjustable Fixing Bar**

Slides up and down to secure wheelchair

**Sliding Buckle**

Slides left and right to fix wheelchair armrests

**Main Vertical Beam**

Aluminium alloy material, high strength and toughness, rust and corrosion resistant

**Wheelchair Gear Lever**

Push against the wheels to prevent the wheelchair from sliding

**Main Switchboard**

Switch on the main power switch to check the battery level

**Charging Hole**


**Non-marking & Non-slipping Tracks**

Extended tracks for smooth climbing

**Directional Wheels for Flat Ground**

Easy to push on flat ground


## Disassembly Steps

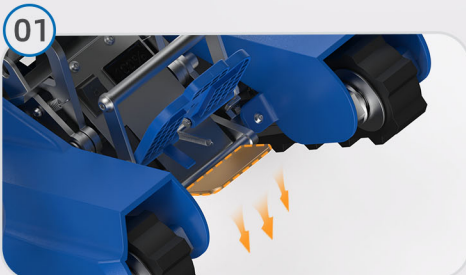

Depress the pedal and release the operating rod




Push the main vertical beam forward to vertical

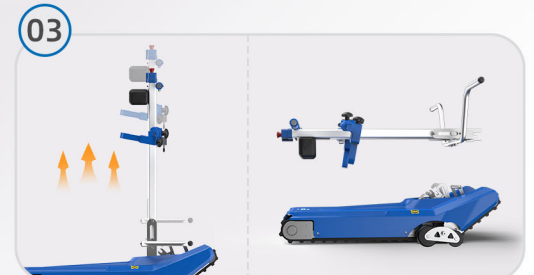

Hold the handle with both hands and lift the main vertical beam to separate the main body

## ■ Push-button Control

Simple operation, easy to use.

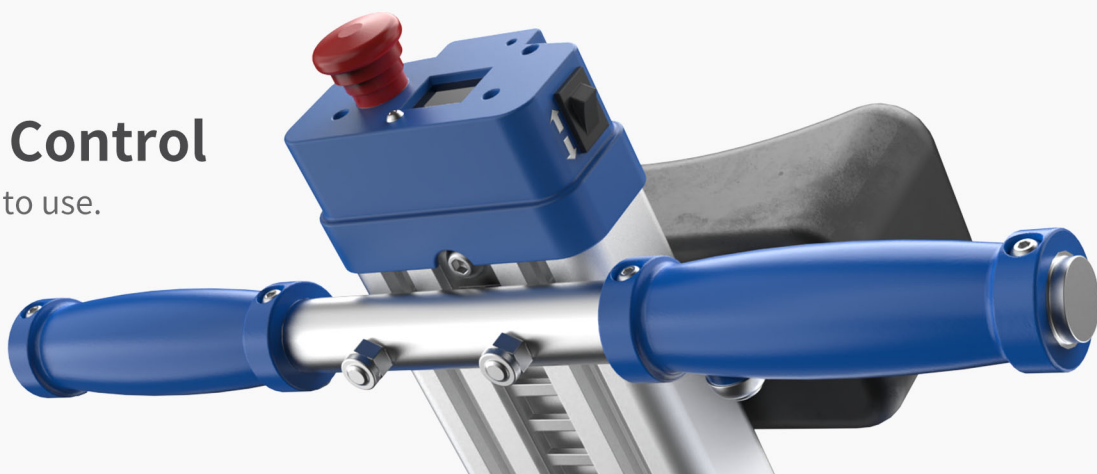


## ■ Crawler Drive Structure

Convenient and safe to go up and down the stairs, smooth operation, comfortable ride.



## ■ Balanced Marking

Reminder to switch between up and down and horizontal modes for safe operation.

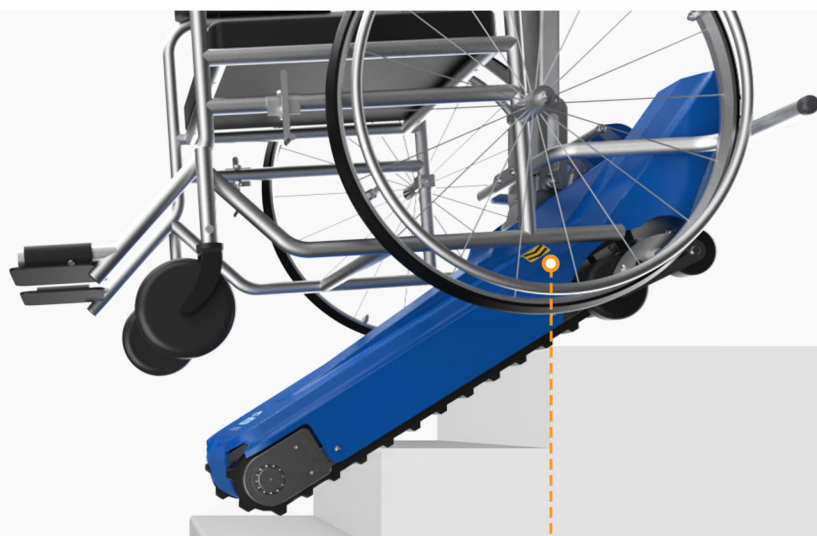

## ■ Removable

The main machine body and operating rod can be quickly disassembled and separated, and easy to carry.





# Parameters



Product Weight  
**55KG**



Maximum Capacity  
**130KG**



Stair Climbing Speed  
**21 Steps/Min**



Battery Endurance  
**46-100 Floors**



Stair Slope  
**<35°**



Battery Capacity  
**25.9V18.2Ah**



Maximum Step Height  
**180MM**



Noise Level  
**<75dB**



IP Grade  
**IPX4**



Minimum Platform Size  
**1700x1700MM**

1320MM



1035MM

1320MM



475MM

1000MM



1535MM






# Wheelchair Installation

## Applicable Wheelchair Size:


- 20-25 inch wheels
- The distance between wheels is 565-760mm



- 01** Push the wheelchair along the bottom shell and stop when the tire touches the gear lever.

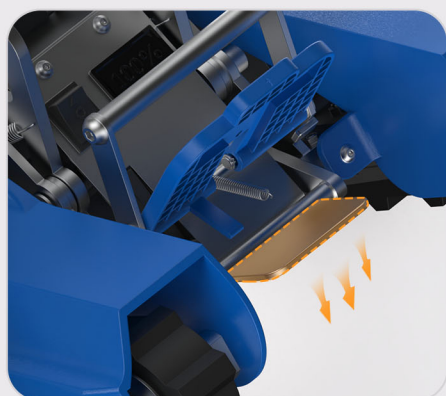


- 02** Depress the pedal and release the lever.

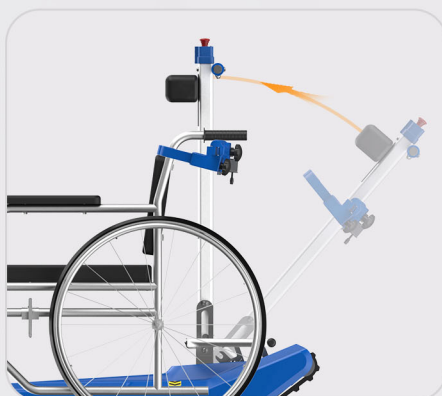


- 03** Push the operating rod forward, parallel to the backrest of the wheelchair.



- 04** Pull the small pull rod outward, and the fixed frame slides until it is close to the bottom of the handle.

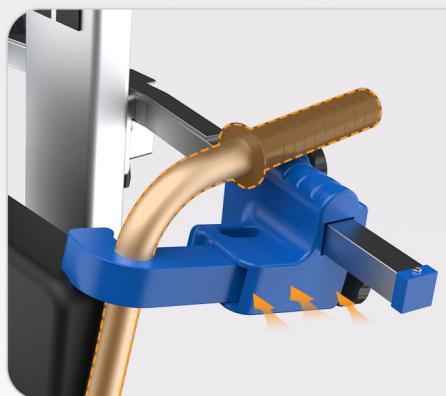

- 05** Adjust the slider so that the handle enters the horizontal clamping hook, and tighten the bolts to fix the slider.

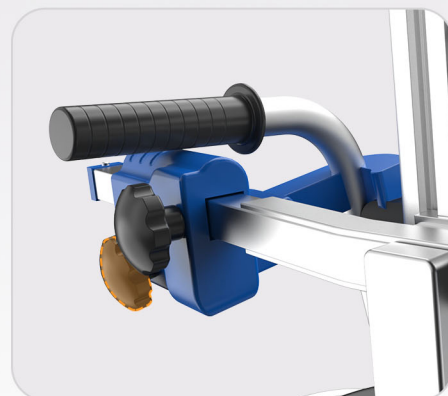

- 06** Tighten the bolts and lock the wheelchair handle.

# Ease Your Way to Anywhere



Community



Office Building



Shopping Mall



Train Station



Subway Station



Airport

**xsto**<sup>®</sup> XSTO CO., LTD.  
Ease Your Way To Anywhere

+86-760-28207258

info@xstomobility.com

xstomobility.com

555 Plant. Ma'an Cun Rd, Cuiheng New District, Zhongshan City, Guangdong, China

